



Fuji Synapse PACS (SSL) - Install Guide

Summary

This documentation provides general instruction for using *Radiology Associate of Ocala's Fuji Synapse PACS (SSL) client*, for referring physicians and hospital use.

System Requirements


The Fuji Synapse PACS (SSL) client is only supported on Windows 7 or 8 running Internet Explorer versions 9, 10, or 11.

*Any other operating systems or browsers not listed above are **NOT supported**, including but not limited to: Mac OS, Unix, Linux, Windows XP or lower, Microsoft Edge, Internet Explorer 8 or lower, Google Chrome, Mozilla Firefox, Opera, and Safari.*

How to Install

To install the *Fuji Synapse PACS SSL client*, use the following steps:

1. In Internet Explorer, open <https://ssl.pacs.raocala.com>
2. You will be prompted to login:

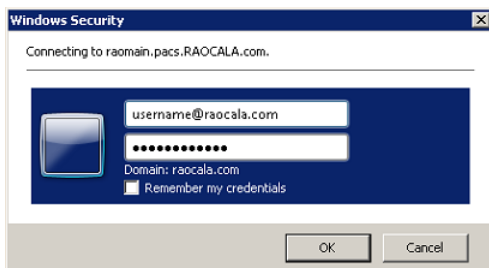

3. Navigate to **Tools > Internet Options > Security** (tab) > **Trusted Sites**, click the **Sites** button, and add: <https://ssl.pacs.raocala.com> to the list of Trusted Sites.
4. Click the **Synapse Workstation Code Download** button:



5. When prompted, click **Run**.
6. Allow the installer to run (for 2-5 minutes) and click **Finish** when the install wizard has completed.